Beingurt Leg Belt

Gebrauchsanleitung · Manual
Bruksanvisning · Käsikirja

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1 Allgemeine Informationen

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of sliding mats.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Leg Belt is part of a product range with a wide range of possible applications. This product is used in particular when transferring seated patients in hospitals and other health care facilities as well as home care settings.

The product may only be used for the specified purpose and under the specified conditions.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: PM-3005 Leg Belt

1.2 Accessories

Item no.: PM-3007 Leg Belt Extension

1.3 Scope of delivery

Leg Belt
User manual

1.4 Storage, transport

Store/transport at -5°C – 40°C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of washing and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

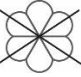
2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Risk of accident! When using the Leg Belt always ensure that the bed, the stretcher or any sort of mobile support whatsoever is standing securely, i.e. that brakes on it are engaged.
- It is essential to ensure that the number of users is sufficient to ensure a safe transfer in the given situation.
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Weight limits must be observed (200 kg).
- Ensure that the Velcro fastener on the belt closes securely during use and verify that the Velcro material is working properly. A belt with defective Velcro material must not be used any longer.
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the sliding mat properties.

2.1 Symbols used

	60°C laundry		max. load capacity see item 6
	Do not bleach		Manufacturer
	Dry at low temperatures		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label
	Do not use fabric softener		

2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

Visual inspection

Ensure that material, seams, handle loops and buckles are undamaged and in functional order.

Anti-slip function

No anti-slip function: product can be moved
Good anti-slip function: Product cannot be moved

➤ **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since the Leg Belt is suitable for all applications involving people who require lower extremity support during transfers.

- To stabilise the transfer in case of reduced muscle tone
- To simplify mobilisation to the edge of the bed
- For significant weight unloading during the transfer
- For greater safety during the transfer

This product may be reused.

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- For individually perceived pain
- For patients who want to walk right away.
- For patients with leg amputations
- For wounds or fractures in the fastening area
- For all clinical pictures for which weight may not be placed on the feet.

3.3 Application risks

- The patient's weight is not on their feet during the transfer → risk of falling
- Patients with perceptual disabilities feel confined → restlessness → risk of falling is possible
- Velcro surface is at the back and doesn't receive enough counter-pressure → risk of falling
- Insufficient pressure applied by the user → difficult movement
- Incorrect positioning of the Leg Belt → difficult movement
- Misuse

3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

3.5 Application illustration

3.6 Application

3.6.1 Transferring the patient

The Leg Belt supports the patient transfer in combination with the Raising Aid Magic or the Support Belt.

The Leg Belt is placed around the legs of the sitting patient from the back to the front at knee height and secured with the Velcro closure. The user places his/her foot between the patient's feet. The user applies pressure to the Leg Belt with his/her knee in order to help raise the patient and to stabilise the turning motion.

3.6.2 Raising the patient in bed

The Leg Belt can assist with raising a recumbent patient out of the bed. To do so, it is placed around the patient's legs and closed from the back to the front with the Velcro closure. Joining the legs can facilitate rotation out of the bed.

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.7 Approved accessories, combination with other products

Accessories optional: Leg Belt extension item no.: PM-3007

The Leg Belt Extension allows the Leg Belt to be extended by 500 mm, from 1400 mm to 1900 mm. Any required length can be achieved with additional extensions. Before using the Leg Belt Extension, please verify that it is safe and suitable for the application.

Leg Belts are used in combination with the following products:

- PM-5080 Turning Disc
- PM-5020/30/50 Sliding Board
- PM-3012/15 Raising Aid Magic/Standard
- PM-6013/14/15 Holding Belt

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

3.7.1 Sets

Item No.	Ingredients set
PM-3025	PM-3015 Raising Aid Magic
	PM-3005 Leg Belt

Please observe the user manuals of the other items as well.

4 Product marking / CE

The Petermann Leg Belt is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

Cleaning	Desinfection (if necessary)
60° washing machine	Recommended disinfectants for wipe disinfection: - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - Bacillo® 30 Foam - Bacillo® 30 Tissues - Bacillo® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues*
Mild detergent	
Do not use cleaning agents with bleaching ingredients or chemical laundry additives! Otherwise the product properties may deteriorate permanently.	
Wash with similar colours only	* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.
Recommended detergent for industrial washing machines: - - Maximo Liquid as single detergent	
Low dryer temperatures (45° - 50° C)	

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-3005	PES/Carbon, latex free	blue/red	200 kg
PM-3007	PES/Carbon, latex free	blue/red	200 kg

Item no.	Description	Dimensions in mm	Weight	UDI
PM-3005	Leg Belt	1000 x 220 x 2,5	345 g	(01)04250355330055(10)x
PM-3007	Leg Belt extension	500 x 220 x 2,5	155 g	(01)04250355330079(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

8 Disposal information

There are no specific disposal regulations for Leg Belts, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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