# Petermann Evakuierungstuch Petermann Evacuation Sheet

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## Table of Contents

### **1 General Information**

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

### **2 Safety information**

- 2.1 Symbols used
- 2.2 Function check

### **3 Using the product**

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
  - 3.6.1 Placement
  - 3.6.2 Evacuation
- 3.7 Approved accessories, combination with other products

### **4 Product marking / CE**

### **5 Cleaning, care and disinfection**

### **6 Technical Data**

### **7 Warranty**

### **8 Disposal information**

# 1 General Information

Thank you for choosing a Petermann product.

## **Please read the user manual carefully before using the product for the first time!**

The Evacuation Sheet was developed for use in nursing care beds to facilitate the fast, safe and gentle evacuation of people who are unable to walk.

According to current understanding in the field of safety, nonambulatory patients in hospitals or other health care facilities have a significantly higher risk of accidents in the event of a fire.

Since elevators may not be used in the event of a fire, nonambulatory patients have a higher risk of suffering severe burns and smoke inhalation. With the use of Evacuation Sheets, it is also possible to move patients to safety via the stairs or windows (ground level) with the assistance of a nurse or emergency responder. Therefore, equipping nursing care beds with Evacuation Sheets serves as a passive fire safety measure. The risk of not being able to evacuate nonambulatory patients in the event of a fire is reduced considerably through the use of Evacuation Sheets.

The Evacuation Sheet is flame retardant and, thanks to its low-friction surface, is designed to enable people of average strength to evacuate a nonambulatory patient. The emergency responder does not need to lift and carry, the movement is done by pulling over floors and stairs. After successful evacuation, the nonambulatory patient can continue to be cared for without any problems, as he is on a mattress with a blanket.

The Evacuation Sheet supports the emergency responders in their working and safety conditions even in extreme stress situations.

The information in this user manual is for the patient who is being evacuated, as well as the user performing the evacuation.

The product may only be used for the specified purposes and conditions.

Contact us if you require additional information or assistance using the Petermann product.

## 1.1 Product versions

**Item no.:** PM-EVA-7053-H Evacuation Sheet

**Item no.:** PM-EVA-7053-S Evacuation Sheet (custom size)

## 1.2 Accessories

No accessories are available for this device.

## 1.3 Scope of delivery

Evacuation Sheet

User manual

## 1.4 Storage, transport

Store/transport at -5 C – 40 C

Other: protect from sunlight

## 1.5 Service life / lifecycle

If it is stored properly, the Evacuation Sheet can be used for at least 15 years (provided it has not been used in a rescue situation). **Evacuation Sheets which have been used for an evacuation may not be reused!**

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

# 2 Safety information

**WARNING indicates a potential danger which could lead to personal injury or material damage.**

- Defective products may no longer be used.
- Weight limits must be observed (200 kg).
- Evacuation Sheets are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- Evacuation Sheets which have been used for an evacuation may not be reused!
- The evacuation mat must not be placed in a dryer or tumble dryer.
- Don't use any cleaning agents which are abrasive or contain chlorine!
- Always allow the product to dry completely after any treatment!
- Evacuation sheets may only be used as transfer aids in accordance with the intended purpose stated in the respective manual. Any other, unrelated use is not permitted.
- Fire safety specifications must be taken into account.
- Fire drills must include the evacuation sheet.
- The following points should be considered in advance during the fire drill, as the fastest possible action is required in the event of a rescue:
  - An assessment of the person to be moved is required.
  - An assessment of the physical abilities of the caregiver is required.
  - An assessment of the local situation is required, does the evacuation sheet fit the spatial conditions.
  - Weight limits must be adhered to.
  - Compliance with the specifications for back-friendly working.
  - If the movement is too fast or jerky, there is a risk of falling.
  - If the buckles are not properly closed, there is a risk of accidents.
- Incorrect operation may occur when used by untrained personnel, therefore instruction is required.
- The evacuation sheet must dry completely after cleaning or disinfection => risk of moisture and mould formation under the mattress.
- - The identification flag must be visibly displayed under the mattress to indicate it to the rescuer.
- - Storage instructions must be followed.

## 2.1 Symbols used

	Wipe off		60°C laundry (wash in laundry net)
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label
	max. load capacity see item 6		

## 2.2 Function check

Inspect the equipment regularly and carefully, in particular before each use.

 <p><b>Visual inspection</b> Ensure that material, seams, handle loops and buckles are undamaged and in functional order.</p>	 <p><b>Load test handles</b> Pull firmly on the handle loops - Check material and seams</p> <p><b>Load test buckles</b> Pull firmly on the closed buckle - Check buckle and seams</p>
--	--

 ➤ **Defective products may no longer be used!**

## 3 Using the product

### 3.1 Indication and further use

The indications cannot be tied to a specific clinical picture. The Evacuation Sheet is suitable for evacuating all nonambulatory patients.

- Rescuing patients from danger zones
- Rapid evacuation → only one person per patient is required
- Safe evacuation for everyone involved

**This product may be reused.**

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning and disinfection of the product. (see item 5)

### 3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- Indications involving a risk of self-harm → Risk of strangulation due to straps

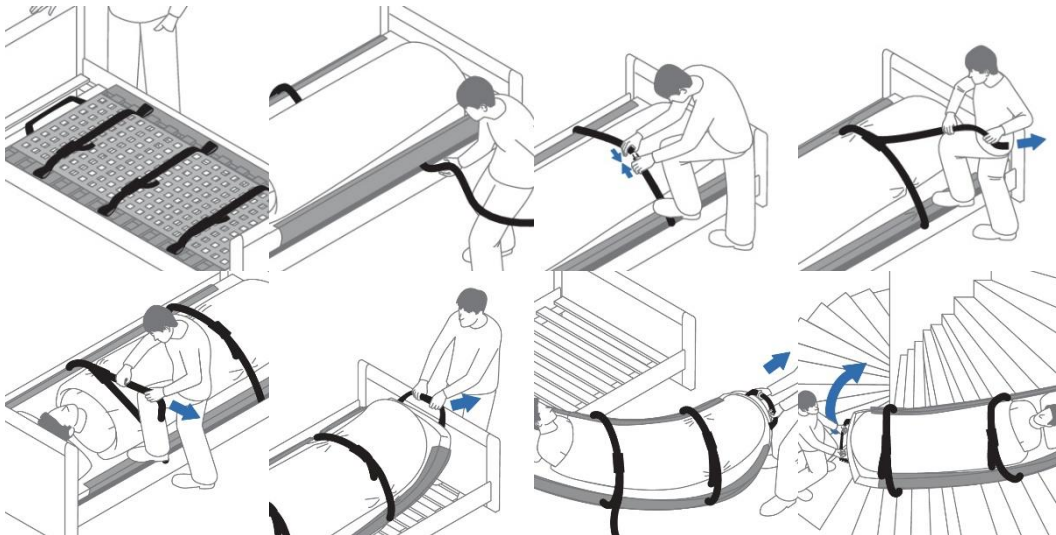
### 3.3 Application risks

- Evacuation Sheet placed in bed incorrectly → difficult evacuation
- Users are not sufficiently trained → faulty/difficult evacuation
- Evacuation Sheet is placed in the bed when wet → Risk of mould formation

### 3.4 Areas of application

The product can be used in hospitals and other health care facilities.

### 3.5 Application illustration

### 3.6 Application

#### 3.6.1 Placement

- Place the Evacuation Sheet directly onto the bed frame in the middle of the bed.
- The straps with clips are tucked in and each is secured with a rubber band.
- **The male ends of the strap clips must be placed on the accessible side of the bed!**
- Place the mattress on the Evacuation Sheet.
- The strap clips are covered by the mattress.
- Use the corner loops to fasten the Evacuation Sheet to the mattress.
- Ensure that the indicator on the clip remains visible → Bed indicator

#### 3.6.2 Evacuation

- If possible, place the head and foot in the recumbent position.
- Place a blanket over the recumbent person.
- Ensure that the arms are close to the body!
- Remove the male ends of the clips from the holders.
- Attach to the matching strap on the other side.
- Pull the straps as tight as possible on the loose end → a protective U-shape is created for the recumbent person.
- Pull the Evacuation Sheet out of the bed by the large handle strap at the foot of the bed.
- The recumbent person can be moved over the floor to a fire safe area.
- Stairs can be negotiated without a problem.
- The Evacuation Sheet is suitable for ground level window rescues.
- The Evacuation Sheet can be carried by up to six helpers.

Visit our website at [www.pm-med.de](http://www.pm-med.de) or watch our comprehensive product videos on our YouTube channel.

### 3.7 Approved accessories, combination with other products

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

## 4 Product marking / CE

The Petermann Evacuation Sheet is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

## 5 Cleaning, care and disinfection

Cleaning	Desinfection (if necessary)
Damp cleaning	Recommended disinfectants for wipe disinfection: <ul style="list-style-type: none"> <li>- Trionic® (DGHM/VAH approval, further information on request)</li> <li>- Mikrobac® Tissues*</li> <li>- Mikrobac® forte*</li> <li>- Bacillo!® 30 Foam</li> <li>- Bacillo!® 30 Tissues</li> <li>- Bacillo!® 30 Sensitive Tissues</li> <li>- Mikrobac® Virucidal Tissues*</li> </ul>
Warm water / mild detergent	
Do not use abrasive or bleaching agents!	
Do not use sharp-edged objects!	* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.
60° washing machine	
Mild detergent	
Do not use cleaning agents with bleaching ingredients or chemical laundry additives! Otherwise the product properties may deteriorate permanently.	

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

Please note that specific hygiene regulations apply to the use of underlays beneath mattresses, such as the Evacuation Sheet (see § 36 of the German Infectious Disease Control Act, Hygiene plan – preventing dampness, mould formation). The Evacuation Sheet may be autoclaved (individually) at max. 70 °C.

The Evacuation Sheet must not be placed in a dryer or tumble dryer. Always allow the product to dry completely after any treatment!

## 6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-EVA-7053-H	PVC-coated polyester weave, with ventilation openings, latex-free	white	200 kg
PM-EVA-7053-S	PVC-coated polyester weave, with ventilation openings, latex-free	white	200 kg

Item no.	Description	Dimensions in mm	Weight	UDI
PM-EVA-7053-H	Evacuation Sheet	2000 x 870 x 4 mm for bed size: Length 1900 x 2050 Width: 800 – 1000 mm	1710 g	(01)04250355370532(10)x
PM-EVA-7053-S	Evacuation Sheet	2000 x 1000 x 4 mm for bed size: Length: 1900 x 2050 Width: 1000 – 1400 mm	1920 g	(01)04250355370549(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

## 7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 10 years and can be extended up to 20 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

## 8 Disposal information

There are no specific disposal regulations for Evacuation Sheets, they can be disposed of in accordance with applicable national and local regulations.

**Petermann GmbH**  
**Waldweg 3**  
**D-91601 Dombühl**  
**Phone: +49 (0) 9868-9339-0**  
**Email: petermann@pm-med.de**

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