Röntgenraupe X-Ray Cassette Placer

Gebrauchsanleitung · Manual
Bruksanvisning · Käsikirja · Mode d'emploi

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1 Allgemeine Informationen

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of sliding mats.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

X-Ray Cassette Placers are the perfect aid for safe and gentle handling of X-ray cassettes on the patient. The innovative principle of the rotating Petermann X-Ray Cassette Placer, where the X-Ray Cassette Placer is simply slid under the patient, is a significant improvement in the use of X-ray cassettes on patients. In order to meet the strict hygiene requirements, the Petermann X-Ray Cassette Placer can easily be wipe disinfected. X-Ray Cassette Placer sets also include a disposable Protection Foil for the X-ray cassette to facilitate a safe and efficient workflow.

The product may only be used for the specified purpose and under the specified conditions.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Option A

Item No.: PMF-8260 X-Ray Cassette Placer 410 x 535 mm

Item No.: PMF-8265 X-Ray Cassette Placer set 3 parts

Item No.: PMF-8266 X-Ray Cassette Placer set 6-parts

Option B

Item No.: PMF-8360 X-Ray Cassette Placer 470 x 535 mm

Item No.: PMF-8365 X-Ray Cassette Placer set 3 parts

Item No.: PMF-8366 X-Ray Cassette Placer set 6-parts

1.2 Accessories

Item No.: PMF-8261 Protection Foil

Item No.: PMF-8361 Protection Foil

1.3 Scope of delivery

Option A 3-parts

2 x X-Ray Cassette Placer 410 x 535 mm

1 x Protection Foil

2 x spreader

User manual

Option A 6- parts

5 x X-Ray Cassette Placer 410 x 535 mm

1 x Protection Foil

5 x spreader

User manual

Option B 3- parts

2 x X-Ray Cassette Placer 470 x 535 mm

1 x Protection Foil

2 x spreader

User manual

Option B 6- parts

5 x X-Ray Cassette Placer 470 x 535 mm

1 x Protection Foil

5 x spreader

User manual

1.4 Storage, transport

Store/transport at -5°C – 40 C

Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected service life is **about two years or respectively 1000 uses**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the use and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Ensure that the uses are performed by the users in a back-friendly manner.
- Weight limits must be observed (200 kg).
- Never leave the patient lying on the X-Ray Cassette Placer unattended. Ensure that the patient is in a safe position!
- Be sure to remove the Protection Foil after every transfer.
- The Protection Foil must not be reprocessed. Reprocessing of the foil can lead to material damage and thus result in harm to the patient.
- Use the spreader (accessory for the cleaning and disinfection process) after cleaning / disinfection to ensure proper drying.

2.1 Symbols used

	Wipe off		max. load capacity see item 6
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label

2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

Visual inspection

Ensure that material, seams, handle loops and buckles are undamaged and in functional order.

Sliding properties

Place the product on a firm surface
Pull your hands over the product with pressure
Good glide - easy gliding
Low glide: no gliding/moving possible

- **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since the X-Ray Cassette Placers are suitable for all applications of X-ray cassettes on patients. The product is used for:

- Simple, back-safe placement of X-ray cassettes under patients with limited mobility.
- Placement of the X-ray cassette under sedated patients in a manner gentle on joints and tissue.
- Placement of the X-ray cassette under comatose patients in a manner gentle on joints and tissue.
- Placement of the X-ray cassette under pain patients in a manner gentle on joints and tissue.
- Reducing the risk of luxation => patient doesn't have to be lifted.

The product X-Ray Cassette Placer may be reused.

X-Ray Cassette Placers are intended for reuse. It is not a single use product. The associated Protection Foil is a single use product and must not be reprocessed. Reprocessing the Protection Foil can lead to material damage and thus to damage to health of the patient

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (item 2), the product may not be used if the patient has any of the following conditions:

- Patients who cannot be X-rayed lying down



3.3 Application risks

- Incorrect technique → high effort despite aids

3.4 Areas of application

The product can be used in hospitals and other health care facilities which are using X-ray machines.

3.5 Application illustration

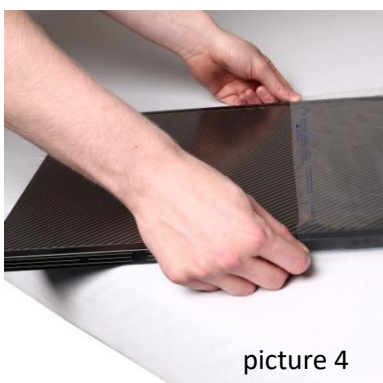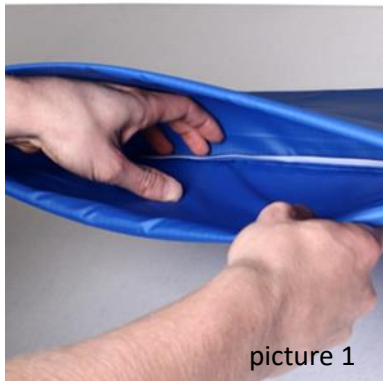3.6 Application

Picture 1

- Reach into the X-Ray Cassette Placer
- Feel for the middle seam

Picture 2

- Pull the middle seam back until your knuckles are visible.

Picture 3

- Ensure that the pad is wrinkle-free.
- Position the X-Ray Cassette Placer according to the required image section.
- Push the X-Ray Cassette Placer as far as possible under the patient, without cramming or lifting the patient.
- Ensure that the fitted sheet is tight.
- Check that the X-Ray Cassette Placer can move freely => Place both hands flat into the opening of the X-Ray Cassette Placer and push forward a little bit.

Picture 4

- Remove the Protection Foil from the dispenser box.
- Pull the Protection Foil over the X-ray cassette.
- The Protection Foil has excess length which must not be folded under => This can be used after imaging to pull the X-ray cassette out of the X-Ray Cassette Placer with both hands.

Picture 5

- Push the X-ray cassette into X-Ray Cassette Placer => until the resistance of the seam can be felt.
- The open side of the foil pouch faces the user.
- The X-ray cassette is placed into the X-Ray Cassette Placer at a slight angle relative to the mattress.
- Push the X-ray cassette under the patient => until the required position is reached.
- Adjust the angle according to the underlying bed material!

Picture 6

- After imaging => Pull the X-ray cassette out of the X-Ray Cassette Placer using the Protection Foil.

Picture 7

- Turn the X-Ray Cassette Placer out from under the patient with a slight turning motion.

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.7 Approved accessories, combination with other products

Accessories

- PMF-8261 Protection Foil
- PMF-8361 Protection Foil
 - The Protection Foil for the X-ray cassette is particularly well suited to rapid and particularly hygienic transfer use with the X-Ray Cassette Placer and integrates seamlessly into the material administration logistics of a facility. The Protection Foil encloses the X-ray cassette and protects it against any contamination.
 - The Protection Foil is a disposable product and may only be used for one person. The Protection Foil is not intended for processing or reuse and may not be washed or treated.
- PMF-8268 Spreader for X-Ray Cassette Placer
- PM-8010 Bed Wedges → recommended for difficult circumstances!

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!


Sets

Artikel-Nr. Set	Bestandteile Set
PMF-8265-1.5	PMF-8260 X-Ray Cassette Placer 2 x
	PMF-8268 Spreader for X-Ray Cassette Placer
	PMF-8261 Protection Foil
PMF-8365-1.5	PMF-8360 X-Ray Cassette Placer 2 x
	PMF-8268 Spreader for X-Ray Cassette Placer
	PMF-8361 Protection Foil
PMF-8266-1.5	PMF-8260 Röntgenraupe 5 x
	PMF-8268 Spreader for X-Ray Cassette Placer
	PMF-8261 Protection Foil
PMF-8366-1.5	PMF-8360 X-Ray Cassette Placer 5 x
	PMF-8268 Spreader for X-Ray Cassette Placer
	PMF-8361 Protection Foil

4 Product marking / CE

The Petermann X-Ray Cassette Placer is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

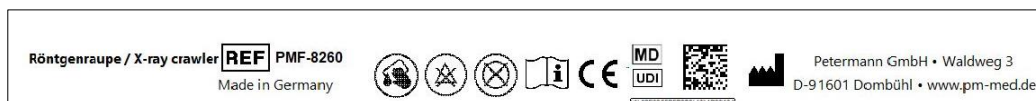5 Cleaning, care and disinfection

Cleaning	Disinfection (if necessary)
Damp cleaning	Recommended disinfectants for wipe disinfection: <ul style="list-style-type: none"> - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - Bacillo!® 30 Foam - Bacillo!® 30 Tissues - Bacillo!® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues*
Warm water / mild detergent	
Do not use abrasive or bleaching agents!	
Do not use sharp-edged objects!	* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PMF-8260	Polyamide, latex free	blue	200 kg
PMF-8360	Polyamide, latex free	blue	200 kg
PMF-8261	LDPE	transparent	N/A
PMF-8361	LDPE	transparent	N/A
PMF-8268	Polypropylene	yellow	N/A

Item no.	Description	Dimensions in mm	Weight	UDI
PMF-8260	X-Ray Cassette Placer	410 x 535x 7	1700 g	(01)04250355352606(10)x
PMF-8360	X-Ray Cassette Placer	470 x 535 x 7	2000 g	(01)04250355353603(10)x
PMF-8261	Protection Foil	450 x 800 x 0,05 mm 100 piece/roll	4020 g	(01)04250355382610(10)x
PMF-8361	Protection Foil	510 x 800 x 0,05 mm 100 piece/roll	4080 g	(01)04250355383617(10)x
PMF-8268	Spreader	55,8 x 35	15 g	(01)04250355382689(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 2 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.
- The **Protection Foil is a disposable product** and the warranty covers proper functionality for one-time use.

8 Disposal information

There are no specific disposal regulations for X-Ray Cassette Placer, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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