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## Table of Contents

### **1 General Information**

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

### **2 Safety information**

- 2.1 Symbols used
- 2.2 Function check

### **3 Using the product**

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

### **4 Product marking / CE**

### **5 Cleaning, care and disinfection**

### **6 Technical Data**

### **7 Warranty**

### **8 Disposal information**

# 1 General Information

Thank you for choosing a Petermann product.

## **Please read the user manual carefully before using the product for the first time!**

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of Raising Aids.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

Raising Aids are part of a product range with a wide range of possible applications. This product is used in particular when raising and/or transferring patients in hospitals and other health care facilities as well as home care settings.

The product may only be used for the specified purpose and under the specified conditions.

The Raising Aids are safe transfer aids, which allow for transfers which are pleasant and comfortable for the patient as well as being simple and back-friendly for the user.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

The Petermann Transfer Aid Magic is patented under following patent no.:

- DE 10 2009 019 716 B3
- EP 2 248 502

## 1.1 Product versions

**Item no.: PM-3015 SOFT** Raising Aid Magic

**Item no.: PM-3012 SOFT** Raising Aid Standard

## 1.2 Accessories

**Item no.: PM-3010** Looped Belt

## 1.3 Scope of delivery

Raising Aid Magic/Standard  
User manual

## 1.4 Storage, transport

Store/transport                    at -5°C – 40°C  
Other:                                protect from sunlight

## 1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of washing and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).


# 2 Safety information

**WARNING indicates a potential danger which could lead to personal injury or material damage.**

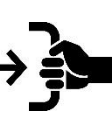
- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Risk of accident! When using the Raising Aid, always ensure that the bed, the wheelchair or any sort of mobile support whatsoever is standing securely, i.e. that brakes on it are engaged.
- It is essential to ensure that the number of users is sufficient to ensure a safe transfer in the given situation.
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Weight limits must be observed (200 kg).
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the sliding mat properties.
- Use the safety loop belt exclusively for short-term patient transfer and only in combination with holding belt and Raising Aid Magic/Standard.
- Mind the possible hazards of tripping and getting caught by the belt systems!
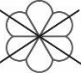
## 2.1 Symbols used

	60°C laundry		max. load capacity see item 6
	Do not bleach		Manufacturer
	Dry at low temperatures		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label
	Do not use fabric softener		

## 2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

	<b>Visual inspection</b> Ensure that material, seams, handle loops and buckles are undamaged and in functional order.		<b>Stability test</b> Bend product to test stability
	<b>Anti-slip function</b> No anti-slip function: product can be moved Good anti-slip function: Product cannot be moved		<b>Load test handles</b> Pull firmly on the handle loops - Check material and seams <b>Load test buckles</b> Pull firmly on the closed buckle - Check buckle and seams

➤ **Defective products may no longer be used!**

## 3 Using the product

### 3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since Raising Aids are suitable for all applications involving people who require assistance.

#### **This product may be reused.**

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

## 3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- Individually perceived pain during the transfer
- Extreme stretching traumas
- Diagnoses for which "sitting" isn't possible.

## 3.3 Application risks

For diagnoses which don't allow for the use of an additional Leg Belt (e.g. types of paralysis), the patient may slide forward due to insufficient support.


Please contact us in these cases, so that we can offer you an alternate solution.

## 3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

## 3.5 Application illustration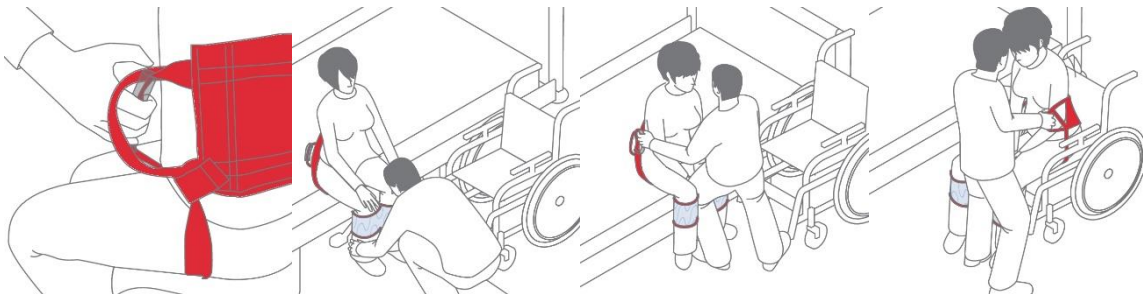

## 3.6 Application

Raising Aids are universal aids with many potential applications.

### **Prerequisites for application without leg belt:**

Adequate tone of the lower extremities and adequate trunk stability must be given!

### 3.6.1 Raising Aid Standard (without flexible additional belt)

- The Raising Aid Standard is positioned as low as possible on the pelvis.
- Grasp the belts
- Step forward
- Shift weight to the back while providing assistance to stand up (figure 2)

## 3.6.2 Raising Aid Magic with flexible additional belt and leg belt

Patients with:                      Little upper body tone  
    Little muscle tone in the legs

### **Placement option A**

The patient can keep their upper body stable on their own and has no high-friction materials underneath them

- Placing the Leg Belt (also see the Leg Belt user manual)
- Place the Raising Aid Magic behind the patient (Figure 1)
- Place the flexible additional belt against the buttocks
- Move it under the buttocks using the attached belt wings

### **Placement option B**

The patient cannot keep their upper body stable on their own and/or has a high-friction material underneath them

- Placing the Leg Belt (also see the leg belt user manual)
- Place the Raising Aid Magic behind the patient (Figure 1)
- Steady the upper body and tilt slightly to the side
- Pull the belt under the buttocks with the free hand
- Change sides
- Repeat the process until the flexible belt is in the desired position

### **Transfer:**

- The height of the back piece can be adjusted as necessary.
- Grasp the belt with wrapping (Figure 1)
- Step forward (Figure 2)
- Shift weight to the back while providing assistance to stand up
- Use small steps to steer in the desired direction
- Step forward
- Tense, assist with safe sitting down (Figure 3)

The aids should be removed after the transfer is complete!

The Raising Aids provide support on the patient's back. This application ensures that the patient leans forward in a physiological manner. With conventional application methods, little attention is paid to leaning forward with a natural motion.

Visit our website at [www.pm-med.de](http://www.pm-med.de) or watch our comprehensive product videos on our YouTube channel.


## 3.7 Approved accessories, combination with other products

### **Accessories optional: Looped belt item no.: PM-3010**

The Looped Belt enables also especially the safe transfer for patients that sit in very narrow wheel chairs or seat pans that prevent applying the flexible additional belt of the Raising Aids under the patient. Both loops of the

Looped Belt have to be attached to the grips of the holding belt and with help of the clip locks then applied under the patient's legs.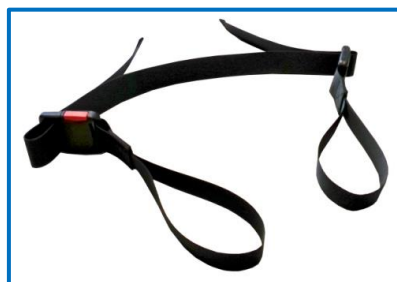

### Combination with other products

Raising Aids are used in combination with the following products:

- PM-3005 Leg Belt
- PM-5080 Turning Disc
- PM-6013/14/15 Holding Belt S/M/L
- PM-5020/30/50 Sliding Board

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

## 4 Product marking / CE

The Petermann Raising Aid Standard/Magic is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

## 5 Cleaning, care and disinfection

Cleaning	Disinfection (if necessary)
60° washing machine	Recommended disinfectants for wipe disinfection: <ul style="list-style-type: none"> <li>- Trionic® (DGHM/VAH approval, further information on request)</li> <li>- Mikrobac® Tissues*</li> <li>- Mikrobac® forte*</li> <li>- Bacillo!® 30 Foam</li> <li>- Bacillo!® 30 Tissues</li> <li>- Bacillo!® 30 Sensitive Tissues</li> <li>- Mikrobac® Virucidal Tissues*</li> </ul>
Mild detergent	
Do not use cleaning agents with bleaching ingredients or chemical laundry additives! Otherwise the product properties may deteriorate permanently.	
Wash with similar colours only	* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.
Recommended detergent for industrial washing machines: <ul style="list-style-type: none"> <li>- - Maximo Liquid as single detergent</li> </ul>	
Low dryer temperatures (45° - 50° C)	

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

The inner reinforcing components must be removed!

## 6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-3015 SOFT Magic	Polyamide, latex free	red/blue	200 kg
PM-3015 SOFT Standard	Polyamide, latex free	red/blue	200 kg
PM-3010	Polyester	black	

Item no.	Description	Dimensions in mm	Weight	UDI
PM-3015 SOFT	Raising Aid Magic	600 x 200 x 10	590 g	(01)04250355330154(10)x
PM-3012 SOFT	Raising Aid Standard	600 x 200 x 10	550 g	(01)04250355330123(10)x
PM-3010	Looped belt	1100 x 30 x 0,2	180 g	Not applicable

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

## 7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

## 8 Disposal information

There are no specific disposal regulations for Raising Aids, they can be disposed of in accordance with applicable national and local regulations.

**Petermann GmbH**  
**Waldweg 3**  
**D-91601 Dombühl**  
**Phone: +49 (0) 9868-9339-0**  
**Email: petermann@pm-med.de**

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