# Wendelaken Turn Sheet

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EN

## **1 General Information**

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

## **2 Safety information**

- 2.1 Symbols used
- 2.2 Function check

## **3 Using the product**

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

## **4 Product marking / CE**

## **5 Cleaning, care and disinfection**

## **6 Technical Data**

## **7 Warranty**

## **8 Disposal information**

# 1 General Information

Thank you for choosing a Petermann product.

**Please read the user manual carefully before using the product for the first time!**

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of sliding mats.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The inner surface of the Turn Sheet is made from particularly low friction silk, which allows for easy movement. The outer surfaces are made from breathable cotton which can remain underneath the patient.

The Turn Sheet is part of a product line which is used specifically to aid in the transfer of patients in home care and other health care facilities.

The product may only be used for the specified purpose and under the specified conditions.

The Turn Sheet is a safe transfer aid, which allows for a transfer which is pleasant and comfortable for the patient as well as being simple and back-friendly for the users.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

## 1.1 Product versions

**Item No.: BM-1600** Turn Sheet (4 parts)

Includes:

Item No.: BM-1600.3 (Top Sheet)

Item No.: BM-1600.4 (Fitted Sheet)

Item No.: PM-7010 (2 parts Non-Slip Mat)

## 1.2 Accessories

**Item No.: PM-7010** (Non-Slip Mat)

## 1.3 Scope of delivery

Turn Sheet (3 parts)

Includes:

1 x Top Sheet

1 x Fitted Sheet

2 x Non-Slip Mat

User manual

## 1.4 Storage, transport

Store/transport at -5 C – 40 C

Other: protect from sunlight

## 1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **three years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of washing and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

# 2 Safety information

**WARNING indicates a potential danger which could lead to personal injury or material damage.**

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- The smooth surfaces significantly reduce resistance due to friction during repositioning. Check the transfer surfaces carefully to ensure they are fixed in place securely (e.g. wheelchair, bed).
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Weight limits must be observed (180 kg).
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the sliding mat properties.

## 2.1 Symbols used

	60°C laundry		max. load capacity see item 6
	Do not bleach		Manufacturer
	Dry at low temperatures		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label

## 2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

 <p><b>Visual inspection</b> Ensure that material, seams, handle loops and buckles are undamaged and in functional order.</p>	 <p><b>Anti-slip function</b> No anti-slip function: product can be moved Good anti-slip function: Product cannot be moved</p>
 <p><b>Sliding properties</b> Place the product on a firm surface Pull your hands over the product with pressure Good glide - easy gliding Low glide: no gliding/moving possible</p>	
 <p>➤ <b>Defective products may no longer be used!</b></p>	

## 3 Using the product

### 3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since Turn Sheets are suitable for all applications involving people who require assistance. Thanks to the reduced friction, the capabilities of the patient can be put to use when Turn Sheets are used. The effort required on the part of the user is reduced.

- Reduction of shear and friction forces
- Patients with an increased risk of bleeding due to skin contact
- Patients unable to move on their own
- Patients with elevated or reduced muscle tone
- Can be used with minimal independent patient mobility (e.g. ALS or MS)
- Application in palliative settings

### **This product may be reused.**

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning and disinfection of the product. (see item 5)

## 3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Observe the safety information under item 2.

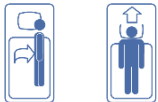
- Independent/active patients
- Wandering patients

## 3.3 Application risks


- Movements are performed too quickly → risk of falling
- Jerky movements → risk of elevated tone
- Patients are miscalculated → risk of falling

## 3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

## 3.5 Application illustration
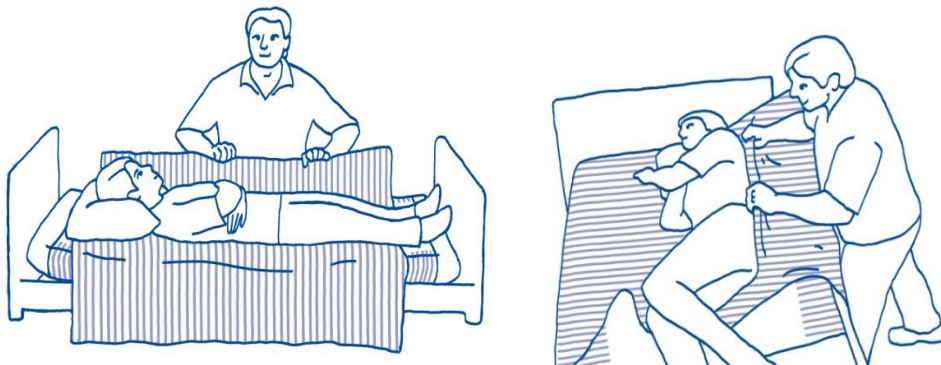
## 3.6 Application

The use of Turn Sheets doesn't involve any prescribed care techniques or methods, it is based on the individual circumstances. The Turn Sheet is a four-piece aid for reducing the friction forces during patient repositioning and movements in the bed. It consists of a Top Sheet (2000 x 1400 mm), a Fitted Sheet (2000 x 900 mm) and two Anti-Slip Mats (255 x 355 mm). The Turn Sheet can remain in the bed permanently.

The Turn Sheet is very versatile. The sliding surface of the Fitted Sheet is finished with extremely low-friction silk. The fact that the silk layers of the Top Sheet and Fitted Sheet lie against one another maximises the reduction of friction forces, producing the same effects as a sliding mat. Placing the Anti-Slip Mats between the upper and lower sliding planes reduces slippage in this area.

### 3.6.1 Applications: Turn Sheet set, 4-piece

The following pieces are used in Turn Sheet applications

- Top Sheet
- Fitted Sheet
- 2 x Anti-Slip Mat

### 3.6.2 Preparation

- Fit the Fitted Sheet onto the mattress
- Tighten it on the sides using the pull cords
- Place the Top Sheet directly on the Fitted Sheet with the sliding surface facing down
- Place the bed sheet and incontinence sheet on the turn sheet as needed

Visit our website at [www.pm-med.de](http://www.pm-med.de) or watch our comprehensive product videos on our YouTube channel.

## 3.7 Approved accessories, combination with other products

Turn Sheets are used in combination with the following products:

- PM-7010/7011, BM-7004 Non-Slip Mats
- PM-8010 Bed Wedges

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

### 3.7.1 Sets

Item No.	Ingredients set
BM-1600	BM-1600.2 Top Sheet
	BM-1600.3 Fitted Shet
	PM-7010 Non-Slip Mat 2 Stück

## 4 Product marking / CE

The Turn Sheet is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

## 5 Cleaning, care and disinfection

Cleaning	Disinfection (if necessary)
60° washing machine	Recommended disinfectants for wipe disinfection: - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - Bacillo® 30 Foam - Bacillo® 30 Tissues - Bacillo® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues*
Mild detergent	
Do not use cleaning agents with bleaching ingredients or chemical laundry additives! Otherwise the product properties may deteriorate permanently.	
Wash with similar colours only	* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.
Recommended detergent for industrial washing machines: - - Maximo Liquid as single detergent	
Low dryer temperatures (45° - 50° C)	

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

## 6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
BM-1600	Cotton/Polyamide, latex free	anthracite/blue	180 kg

Item no.	Description	Dimensions in mm	Weight	UDI
BM-1600.3	Turn Sheet (top sheet)	2000 x 1400 x 1	675 g	(01)04250355316035(10)x
BM-1600.4	Turn Sheet (span sheet)	2000 x 900 x 1	765 g	(01)04250355316042(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

## 7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship
- The warranty period is 2 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning**
- The company Petermann is not liable for accidental or consequential injury or material damage

## 8 Disposal information

There are no specific disposal regulations for Turn Sheets, they can be disposed of in accordance with applicable national and local regulations.

**Petermann GmbH**  
**Waldweg 3**  
**D-91601 Dombühl**  
**Phone: +49 (0) 9868-9339-0**  
**Email: [petermann@pm-med.de](mailto:petermann@pm-med.de)**

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